SANITAS

SFB 09

- D** **Gebrauchsanleitung**
Fuß-Sprudelbad
- GB** **Operating Instructions**
Foot Bubble Spa
- F** **Mode d'emploi**
Bain de pieds bouillonnant
- I** **Istruzioni per l'uso**
Idromassaggio
- RUS** **Инструкция по применению**
Гидромассажная ванна для ног

Serviceadresse:
Hans Dinslage GmbH
Riedlinger Straße 28
88524 Uttenweiler, GERMANY
Tel.-Nr.: +49 (0) 7374-915766
Fax-Nr.: +49 (0)7374-920723
E-Mail: service@sanitas-online.de

D

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ENGLISH

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

General information

Our feet play a very important role throughout our lives as they have to bear our full weight. Each foot has 26 bones, 22 joints and 107 ligaments and they all need care. The choice of a foot bubble bath is a step in the right direction. The gentle vibration massage will relax tense muscles and after just a few minutes you will feel more relaxed and stress-free. This effect will be intensified by the bubble bath.

A tip: adding arnica, rosemary or sea salt extracts to warm foot bath water will stimulate the circulation. Addition of lavender or thyme to lukewarm foot bath water will bring relief to tired feet.

Instructions for use

Fill the foot bubble bath with warm or cold water up to the fill line (approx. 1–2 cm). Ensure that the rotary switch is set to “0” before the unit is connected to the mains.

Make yourself comfortable in a chair and place your feet on the foot rest and apply slight pressure. Never stand up in the foot bath. The rubber feet on the base of the unit prevent it from sliding. Set the desired function using the rotary switch.

- Off (0)
- Vibration massage (1)
- Heat and bubble (2)
- Heat with Bubble and vibration massage (3)

A heating element is fitted below the foot surface so that hot water can be kept hot for some time. However, this heating element is not suitable for heating cold water.

To obtain an optimal effect a massage time of up to approx. 20 minutes is recommended.

After your foot massage, turn the switch to “0” and remove the plug from the mains socket. The drain holes allow the water to be removed.

Cleaning

After use rinse the unit with tap water. The foot bath can be cleaned with a mild, non-foaming household cleaner, e.g. neutral vinegar. To dry the air lines, switch to the bubble setting without water for approx. 1 minute.

Massage Attachments for Massage of the Foot Reflex Zones

Additional comfort is provided by the local massage of the reflex zones. Massage of specific areas on the soles of the feet can exert an effect on organs of the body.

It is recommended that you read the specialised literature and consult a doctor before the reflex zone massage to ensure that you obtain the desired effect.

Important Information – Keep for Later Use

- **Read these instructions carefully and show them to other users.**
- Plug the unit only into the mains voltage specified on the rating plate.
- You must never connect the foot bath to the power supply when your feet are already in the water. If the device is damaged, this can lead to lifethreatening electric shock.
- Ensure that the foot spa is undamaged before using it. If any fault or malfunction should occur during use, switch the unit off immediately and remove the plug from the mains socket. Never attempt to repair the unit yourself. Improper repairs can expose the user to considerable risks.
- If the mains lead of the unit is damaged this must be replaced by a service centre as special tools are required.
- The unit is intended only for the purpose described in the instructions for use. The manufacturer cannot be held responsible for damage caused by improper or careless use. The unit is not intended for commercial use, nor for use in medical clinics etc.
- The unit should not be used for longer than 20 minutes without interruption.
- Never operate the unit unattended.
- Never stand in the foot bath.
- Always switch the foot bath off and remove the plug from the mains socket before moving it or cleaning it. Always make sure your hands are dry.
- Never immerse the device in water.
- Children are not aware of the dangers associated with electrical appliances. Make sure that the foot bath cannot be used by children without supervision.
- To fill the foot bath, it must not be placed in the shower, bath tub or wash basin.
- The device must not be connected to the power supply while being filled.
- Consult your doctor if you have any medical condition which affects the legs or feet, e.g. phlebitis, varicose veins or diabetes.
- The water must be drained only through the drain holes.
- The surface of the appliance is hot, persons who are insensitive to heat must use it with particular care.
- Do not operate the foot massager without water.
- Do not open the foot massager. Failure to comply invalidates the warranty. You must not adjust or repair the foot massager yourself or its correct function will no longer be guaranteed.

Disposal

Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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