# SANITAS

SIL 06 – SIL 16

SIL 06

SIL 16

- ▣ **D** **Gebrauchsanleitung**  
*Infrarotlampe*
- ▣ **GB** **Instruction for Use**  
*Infra-red lamp*
- ▣ **F** **Mode d'emploi**  
*Lampe à infrarouges*
- ▣ **I** **Instruzioni per l'uso**  
*Lampara a raggi infrarossi*
- ▣ **RUS** **Инструкция по применению**  
*Прибор инфракрасного излучения*

Serviceadresse:  
Hans Dinslage GmbH  
Riedlinger Straße 28  
88524 Uttenweiler, GERMANY  
Tel.-Nr.: +49 (0) 7374-915766  
Fax-Nr.: +49 (0)7374-920723  
E-Mail: [service@sanitas-online.de](mailto:service@sanitas-online.de)

## Anwendung

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## ENGLISH

### Application

This infrared lamp is only intended for radiating the human body.

Infrared radiation causes warmth to be transported to the human body. The blood supply in the radiated skin is increased and the metabolic turnover in the field of temperature enhanced. The mode of action of the infrared light encourages the body to heal; healing processes are thus assisted in a specific area.

Infrared light can be used as a parallel treatment in diseases of the ear, nose and throat and to assist in facial and beauty care, especially in case of impure skin.

However, first ask your family doctor whether application in the individual case is advisable from a medical standpoint.

### Use

Remove all packaging material before using the appliance. Before use, check whether the infrared lamp is properly screwed in. Always ensure that the appliance is on a secure and even surface. The distance between the infrared lamp and the radiated part of the body should not be less than the following:

SIL 06: at least 24 inch (60 cm)

SIL 16: at least 32 inch (80 cm)

Always close your eyes during the radiation! The recommended duration of treatment is maximum 12 minutes.

We recommend selecting a shorter radiation period to start with. However, the skin of some people may show an over-sensitive (e.g. with extreme redness, blistering, itching, heavy sweating) or allergic reaction to heat radiation, even when the lamp is used correctly. Examine the radiated section of the body regularly. Discontinue radiation immediately if any signs of an over-sensitive or allergic reaction appear and consult a physician.

The appliance has an adjustable lampshade. The 5 positions of inclination make it possible for you to adjust the infrared lamp according to your needs.

### Cleaning/changing the infrared lamp

Before every cleaning and when exchanging the infrared lamp, the appliance must be switched off, disconnected from the mains and must be cool.

The appliance can be cleaned with a damp cloth. Do not use abrasive cleaners or scouring agents.

Ensure that no water penetrates the appliance. Use the appliance again only when it is completely dry.

Please note that only infrared lamps of the original type may be used as replacements.

<b>Appliance:</b>	<b>Replacement infrared lamp (type):</b>	<b>Specifications:</b>	<b>Explanation of symbols:</b>
SIL 06	Infrared R95 E (100 W)	230 V ~50 Hz	 Note – read operating instructions.
SIL 16	Infrared PAR 38 E (150 W)	230 V ~50 Hz	 Application part Type B
			 Device of protection class II

# ! Important information – ■ Please retain for later use

## **Please read the information carefully and keep it accessible for other users.**

- The lamp housing and the infrared lamp become very hot during operation. There is a risk of burning if these are touched. Always let the lamp cool down first before touching it.
- Remove all packaging material before the appliance is used.
- The appliance may only be connected to the mains voltage mentioned on the type plate.
- While using the lamp on the face, always keep the eyes closed in order to avoid injury to the eyes (do not look directly at the infrared light).
- This device is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety or they are given instructions on how to use the device.  
Children should be supervised to ensure that they do not use the device as a toy.
- Do not use it on persons sensitive to heat.  
The sensitivity to heat can be reduced or increased in the following cases:
  - diabetic patients
  - persons with sleepiness, dementia or difficulties concentrating
  - persons with illness-related skin changes or persons with scarred skin areas in the application area
  - persons with allergies
  - children and older persons
  - after consuming medications or alcohol
- In case of acutely inflammatory processes, radiation should only be carried out after consulting a doctor.
- Always limit the period of use and check the reaction of the skin.
- Medications, cosmetics or food can lead to a over-sensitive or allergic reaction of the skin under certain conditions. Radiation must be ended immediately in this case.
- If the appliance is operated continuously, caution and attention are advisable - never fall asleep during radiation!
- Prolonged use can cause burns to the skin.
- Do not touch the infrared lamp or unscrew it when it is connected to the mains.
- Children do not understand the risks associated with electrical appliances. Make sure the appliance is not used by children without appropriate supervision.
- The appliance may be used only under supervision.
- Do not cover or pack the appliance when it is hot.
- Always pull out the plug from the socket and let the appliance cool before you touch it.
- Inflammable objects must be kept at a distance of at least 1.0 m from the infrared lamp. Do not hang the appliance on the wall or from the ceiling.
- Do not touch the appliance with wet hands when it is connected to the mains; water should not be sprayed on the appliance. Operate it only when it is completely dry.
- Protect the appliance against blows and impact.
- Do not remove the plug from the socket by pulling the cable.
- In case the cable or the casing is damaged, please contact the customer service or dealer, as special tools are required for repairing it.
- Isolation from the mains supply is only guaranteed if the power plug is unplugged from the mains.
- Please note that the mains cable must not be wound round the infrared lamp either for storage or when in use.

## Disposal

Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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