# SANITAS

## SIS 40

- Ⓓ **Gebrauchsanleitung**  
Insektenstichheiler
- ⒼⒷ **Operating Instructions**  
Insect bite healer
- Ⓕ **Mode d'emploi**  
Appareil soulageant les  
piqûres d'insectes
- Ⓔ **Instrucciones para el  
uso**  
Curador de picaduras  
de insectos

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## ENGLISH

### Method of Operation

Insects sting human skin for the most diverse reasons. Mostly, they want to suck blood and thereby also convey toxins. Painful and swollen puncture marks are the result in the case of very many insect bites. Often, these insect bites cause severe itching. The inflammation is often made worse by scratching. Relief has been possible until now by cooling or by using special ointments.

The insect bite healer provides entirely new and highly effective relief in the case of insect bites.

The toxins which can cause itching, swelling and inflammation generally consist of protein molecules. When a specific temperature limit is exceeded, proteins alter their molecule structure and thus often lose their effect. The healing process is often experienced as more pleasant.

The pain receptors (nociceptors) at the point of treatment respond to the effect of the heat. The itching subsides, symptoms improve

and the healing process is often experienced as more pleasant. The insect bite healer uses this effect and warms the skin with the insect bite up to a specific heat limit.

The effectiveness of the insect bite healer has been clinically proven (Source: Dr. med. M. Sorf, Hospital and Polyclinic, Nové Mesto n/V (SK) - 2006):

- 74% of all test persons notice an immediate improvement after the heat treatment.
  - 66% of all test persons feel that the insect bite has 'completely subsided' after 24 hours, a further 20% notice an improvement.
  - With 78% of all test persons, the swelling or reddening is less than 6 mm in diameter after 24 hours.
- By way of comparison: if there is no immediate heat treatment, the proportion is only 18%.

## **Information about the device**

- This unit is intended for use on human beings.
- The device should be stored in such a way as to prevent any inadvertent release of the button. Occasionally burns can occur as well as a severe build-up of heat.
- Only press the control button once the heating plate is in contact with the skin. If you press the button too early and 'heat up' the heating plate, this can cause burns on the skin.
- For reasons of hygiene, the insect bite healer should only be used on one person.
- Diabetics should not use the insect bite healer as their pain sensitivity might possibly be reduced and they might inflict burns on themselves.
- This device should not be used with persons who are helpless, with children under 3 or persons insensitive to heat (e.g. persons with skin changes related to illness or persons with scarred skin in the area of application, after taking any pain-relieving medication, alcohol or drugs).
- The insect bite healer does not help against insect allergies.
- This unit is only intended for personal use and not for commercial applications.

- You should protect the device from impact, moisture, dust, chemicals, major temperature fluctuations and direct exposure to the sun's rays.
- Repairs should only be carried out by the customer service department or by authorized dealers. However, always check the battery first and replace it if necessary prior to making a complaint.
- The device meets the requirements of the European Directive for Medical Electrical Equipment 93/42/EEC.
- This unit is in line with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility (EMC). Please note that portable and mobile HF communication systems may interfere with this unit. For more details, please contact customer service at the address indicated.

### **Advice on dealing with insects**

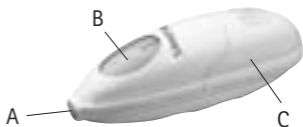
- Do not irritate insects by any rapid or hectic movements. Do not try to swot them.
- Be careful when drinking sweet beverages out of doors, particularly from bottles or cans. Always use a straw!
- Keep well away (3–4 m) from nests and swarms of insects.
- During outdoor activities, always wear close-fitting clothing with long sleeves.
- If possible, do not kill insects as this releases aromas which can make other nearby insects aggressive.
- In living areas, carefully trap individual insects using a glass and a piece of paper and take them outside.
- Fit flyscreens in front of windows and doors.
- Use mosquito nets in sleeping areas.
- Particularly in the case of a known insect allergy, you should be very careful in dealing with insects and avoid any situations where a greater number of insects can be expected. If necessary, inform persons around you about the allergy in order that they can react promptly in the event of an emergency. If necessary, carry an emergency kit with you and explain to other persons how to use it in the event of an emergency.

## Description of Device

A Heating plate

B Control button with indicator lamp

C Battery compartment cover

## Operation

### Inserting Battery

To open the device, slide back the battery cover. A 9 V block battery is provided in the battery compartment on delivery. Take it out and unpack it. Connect the contact clip at the end of the cable to the battery, making sure that the battery polarity is correct. Once the battery has been connected, put it back into the battery compartment. The contact clip should point backwards, with the projecting cables stored underneath the device. When closing the battery cover, make sure that no cables are caught as they might thereby become damaged.

### Changing the battery

If the indicator lamp does not light up any more when the control button is pressed, you have to replace the battery. Use a standard 9 V block battery for this.

The battery provided will last for approx. 100 applications.

Never use rechargeable batteries.

Used batteries do not belong in domestic garbage. You are legally obliged to dispose of the batteries.

Dispose of them through your specialist electrical supplier or at your local recycling point.

Note: Batteries containing pollutants are marked with the following symbols: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury

## Application

The insect bite healer should not be used:

- with children under 3 or with persons who are helpless
- by diabetics or persons insensitive to pain

- when taking any pain-relieving medication, drugs or alcohol
- in areas where the skin is particularly sensitive (e.g. eyelids, mucous membrane, etc.)

Before use, always make sure that there is no longer any insect sting remaining in the puncture wound. Carefully remove the sting before using the device.

Using the device immediately after the insect sting is the most successful, as the effect of the toxins has not yet been able to unfold in the skin. If you wait too long before treating the puncture wound and if it has already started itching or become inflamed and swollen, the insect sting healer can only now work to a limited extent. Treatment with the insect bite healer should nonetheless be repeated. Mostly the itching can be reduced and the healing accelerated.

Place the insect sting healer with the heating plate on the sting to be treated. Now press the button and hold it down throughout the entire application. The translucent blue indicator light of the control button indicates that the heating plate is warming up.

The heating plate takes approx. 25 seconds to reach its maximum temperature of approx. 50 °C. Only keep the device on the puncture wound for as long as it still seems agreeable to you, but never longer than 30 seconds.

The temperature of the heating plate is limited by the electronics of the insect sting healer. The temperature that is reached and also temperature awareness depends on skin type, moisture content, fat content, skin temperature, ambient temperature, etc. Make sure that you stop the application and release the control button as soon as it becomes unpleasant and actually before any pain occurs, as well as before the maximum application time is reached. Otherwise severe burns can occur. The device does NOT have any automatic switch-off function.

As soon as you release the button, the device automatically switches off and the heating plate slowly cools down. If you use it again before the heating plate has entirely cooled down, the maximum application time reduces accordingly. Ideally, before any new application, you should allow the insect sting healer to cool down for 30 seconds.

Wait at least 60 seconds before putting away the device again. The device should be stored in such a way as to prevent any inadvertent release of the button.

## Cleaning and disposal

- Always remove the battery from the device before you clean it.
- Clean the device with a slightly damp cloth. In case of more extreme soiling you can also moisten the cloth with mild soapy water.
- Make sure that no water penetrates into the device. If this should ever happen then only use the device once it is completely dry.
- Do not use any chemical cleaners or abrasive agents for cleaning.
- Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

## Technical Specifications

Model	SIS 40
Power supply	9 V block battery
Weight	37 g (without battery), Alkaline
Max. temperature of heating plates	68 °C +0 °C/-3 °C (without skin contact) approx. 47 °C ±3 °C (with skin contact), value very much depends on skin properties
Warm-up time from room temperature to max. temperature	approx. 25 seconds (with skin contact)
Maximum permissible operating temperature	0 °C to +40 °C, 20-65% relative humidity
Maximum permissible storage temperature	0 °C to +60°C, <95% relative humidity
Explanation of symbols	Application part type B    Read the Instructions for Use

Rights are reserved to make changes to the technical specifications without notice for updating purposes.

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