DEUTSCH

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ENGLISH

with this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use.

Measuring methods:

Temperature measurement in the anus (rectal)

This method of measurement is most reliable and most accurate. Guide the tip of the thermometer carefully 2 to 3 cm into the anus.

Measurement in the mouth cavity (oral)

For this purpose, guide the tip of the thermometer carefully into one of the heat pouches beneath the tongue, to the left or the right of the root of the tongue.

Measurement of temperature in the armpit (axillary)

This method of measurement is relatively inaccurate. Therefore, it cannot be recommended from a medical point of view.

Mode of measurement	Recommended time required for measurement	Mean divergence to the rectal method
in the anus	until the acoustic signal	–
in the oral cavity	until the acoustic signal	-0,4 °C to -1,5 °C
in the armpit	minimum 5 minutes	-0,7 °C to -2,0 °C

Use

First remove the protective foil from the display. For switching on, briefly press the on/off key. A short acoustic signal will announce that the thermometer is switched on. The thermometer will carry out a self-test for about 2 seconds. During the selftest all segments of the display will appear. Then the reference reading (37 °C) appears. Then the measurement symbol „°C“ flashes and „Lo“ is shown on the display. The thermometer is now ready for use. During the measurement, the current temperature is displayed continuously, and the „°C“ symbol flashes on and off. The measurement is completed when a stable temperature has been reached. When this stage has been reached, an acoustic signal is heard, the „°C“ symbol stops flashing and the temperature measured is displayed. As long as the measured temperature is below 32 °C, „Lo °C“ will appear. Higher temperatures will be shown directly, e.g. 32,1 °C. By prolonging the length of measurement until after the acoustic signal, more accurate measurements can be obtained. In case the temperature is measured in the armpit, the thermometer should be left in the armpit for the recommended time of measurement, in spite of the acoustic signal. If temperatures over 37,8 °C are measured, a fever alarm sounds (short beeping tones, 10 seconds long). If temperatures higher than 42,9 °C are measured, the symbol for „Hi °C“ will appear. To prolong the life of the battery, please switch off the thermometer by briefly pressing the on/off key after the measurement. Otherwise the thermometer will automatically switch off after 10 minutes.

Memory

The last measured value will be stored automatically. Press the ON/OFF button to display this value. The thermometer first carries out a self test. Next the last stored measured value is displayed for approx. 2 sec and then the reference value of 37 °C and „Lo“ is shown on the display. The thermometer is ready to take a measurement and the previously stored measurement value is automatically replaced by the new measurement value.

Changing the battery

The clinical thermometer contains a long-lasting alkali-manganese battery (type LR 41, 1.5 V) for approximately 2 years. The battery must be changed when the battery symbol appears. Release the battery compartment cover by gently pressing on the catch. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then close the battery compartment, making sure that the catch clicks into place.

Used batteries do not belong in the regular domestic refuse. You are legally obliged to dispose of the batteries. Dispose of them through your local dealer for electric goods or your local depot for special waste. You are legally obliged to dispose of the batteries. Note: Batteries containing pollutant substances are marked as follows: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. These batteries contain lead (Pb) and mercury (Hg).

Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- **⚠ Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- **⚠ Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

Cleaning/Disinfection

The device must be cleaned and disinfected before and after each use. The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfectant without any difficulties. Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's instructions. Other cleaning agents or methods could impair the function of the thermometer or damage it.

Name of the agent: Isopropanol 70% Pharmacy, Gigasept FF, Lysoformin; Aseptisol.

Storage/Disposal

When you are not using the thermometer, please retain it in the original protective cover. The device must be tested for accuracy after 2 years. Please send the complete device to the dealer's or service address. For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

Technical data

Model no.:	SFT 01/1
Type:	SFT 01/2
Range of measurement:	32 °C to 42,9 °C
Accuracy of measurement:	± 0,1 °C in a water bath with a temperature of 35,5 °C to 42 °C ± 0,2 °C in a water bath with a temperature of 32,0 °C to 35,4 °C ± 0,2 °C in a water bath with a temperature of 42,1 °C to 42,9 °C
Ambient temperature:	+10 °C to +40 °C, with 30 % to 85 % relative humidity
Storage temperature:	-10 °C to +60 °C, with 25 % to 90 % relative humidity

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