## DEUTSCH

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## ENGLISH

### Signs and symbols

The following symbols are used in these instructions for use, on the packaging and on the type plate for the device and accessories:

	Caution
	Follow instructions for use
	Type BF applied part
	Disposal in accordance with EC Directive WEEE (Waste Electrical and Electronic Equipment)
	Manufacturer
<b>IP 27</b>	Protected against foreign objects equal to or greater than 12.5 mm and against temporary immersion in water.
<b>LOT</b>	Batch designation
<b>Storage/Transport</b>	Permissible storage and transport temperature and humidity
<b>Operating</b>	Permissible operating temperature and humidity
<b>CE</b> 0483	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.

**Included in delivery:** • Thermometer • Instruction for Use • 1 x 1,5V LR 41 battery

### Important safety information – retain for later use.

In order to make optimum use of all the merits of the clinical thermometer, you should carefully read the operating instructions before using it, retain the instructions for later use and also keep them accessible for other users.

- The thermometer is intended to measure the temperature of the human body.
- The thermometer is only designed for the measuring area on the human body stated in the instructions for use.
- Children should not use the thermometer without supervision.
- Before each use, check the thermometer for signs of damage or wear. Damaged or worn thermometers should not be used.
- Always wait for the acoustic signal which indicates the minimum time required for measurement. Discuss the temperature with your family physician.
- The thermometer contains sensitive electronic parts. Therefore, protect it from blows, bending, high temperature or direct exposure to sunlight.
- This thermometer has a flexible measuring tip to provide greater comfort and safety during measurement, particularly

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has been reached. An acoustic signal (10 long beeps) sounds, the »°C« (»°F«) symbol stops flashing and the measured temperature is displayed. So long as the temperature is less than 32 °C (89,6 °F), the »°C« (»°F«) symbol continues to flash and »Lo °C (Lo °F) « is shown on the display. Higher values are displayed directly, e.g. 32.1 °C (89,9 °F). If temperatures over 37.8 °C (100 °F) are measured, a fever alarm (30 short beeps in 10 seconds) sounds. If temperatures over 43.0 °C (109,4 °F) are measured, the symbol for »Hi °C (Hi °F)« appears. The measurement time is very quick compared to conventional thermometers and is only 10 seconds for the rectal measurement. More accurate measurements are possible by extending the measurement time beyond the acoustic signal. For measurements in the armpits the recommended measurement time should be maintained in spite of the acoustic signal. To prolong the life of the battery, please switch off the thermometer by briefly pressing the On/ Off key after the measurement. Otherwise the thermometer will automatically switch off after 10 minutes.

### Memory

The last measured value will be stored automatically. This value is displayed when the On/Off button is pressed. The memory symbol is also shown beside the measured value, e.g. 37.5 °CM (99.5 °FM).

#### Changing the measuring unit

The temperature can be displayed in degrees Celsius (°C) and degrees Fahrenheit (°F). To set the measuring unit press and hold the On/Off button for longer than 2 seconds when switching on until the measuring unit changes. The saved value is displayed in the new measuring unit.

#### Changing the battery

The clinical thermometer contains a long-life alkali-manganese battery (LR 41, 1.5 V), which will last for approx. 2 years. The battery must be changed when the battery symbol appears. Press down the battery cover at the end of the device with your fingers and then pull it firmly back. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then put the battery cover back onto the case.

Used batteries do not belong in the regular domestic refuse. Dispose of them through your local dealer for electric goods or your local depot for special waste. You are legally obliged to dispose of the batteries. Note: Batteries containing pollutant substances are marked as follows:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. These batteries contain mercury (Hg) and lead (Pb).

 Pb Cd Hg

#### ⚠ Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, flush out the affected areas with water and seek medical assistance.
- ⚠ **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.

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- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect the batteries from excessive heat.
- ⚠ **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

#### Cleaning/Disinfection

The device must be cleaned and disinfected before and after each use. The thermometer is 100% water-proof. For the purpose of cleaning or disinfection, it can be dipped into water or disinfectant without any difficulties.

Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's instructions. Other cleaning agents or methods could impair the function of the thermometer or damage it.
**Name of the agent:** Isopropanol 70% Pharmacy; Gigasept FF; Lysoformin; Aseptisol.

#### Storage/Disposal

When you are not using the thermometer, please retain it in the original protective cover. If using the device for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

#### Technical data

Typ:	SFT 11/1
Range of measurement:	32 °C (89,6 °F) to 42,9 °C (109,2 °F) ±0.1 °C in a water bath with a temperature of 35.5 °C (95,9 °F) to 42,0 °C (107,6 °F) ±0.2 °C in a water bath with a temperature of 32.0 °C (89,6 °F) to 35,4 °C (95,7 °F) ±0.2 °C in a water bath with a temperature of 42,1 °C (107,8 °F) to 42,9 °C (109,2 °F)
Accuracy of measurement:	10 °C (50 °F) to 40 °C (122 °F), with 30 % to 85 % relative humidity
Storage temperature:	–10 °C (14 °F) to +60 °C (140 °F), with 25 % to 90 % relative humidity

Ambient temperature:	
Storage temperature:	

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NU Service GmbH